

Tuttnauer Chamber Brite™

Powdered Autoclave Cleaner

**No Mess
No Hassles
No Measuring**

Chamber Brite powder is simply the easiest way to keep autoclaves clean and free of water "Lime Scale" deposit build-up. Designed specifically for steam sterilizers, Chamber Brite effectively cleans and descales the reservoir, chamber, and copper tubing within minutes.

- Convenient, pre-measured packets
- Cleans reservoir, chamber, and copper tubing
- Uses less storage space
- Replaces messy liquid cleaners



tuttnauer
Sterilization & Infection Control Products

Chamber Brite™

Powdered Autoclave Cleaner

Few simple steps to a hassle-free clean



Cleaning Instructions:*

1. Sprinkle Chamber Brite powder along the bottom of the **COLD** autoclave chamber.



2. Start a sterilization cycle **WITH WATER and NO DRYING CYCLE**



3. At the end of the cycle drain solution from reservoir and discard.



4. Fill reservoir and repeat steps 2 and 3 without Chamber Brite.



5. Wipe inside of chamber with damp rag or sponge.

6. Fill reservoir with distilled water, and sterilizer is ready for use.



For best results, clean autoclave every 20 cycles or once per week as needed. It is essential to follow the equipment manufacturer's directions for cleaning and maintenance.

*For full instructions on how to use Chamber-Brite -follow the directions on the box.

Tuttnauer's Table-Top Autoclaves



Fully Automatic EA/EKA Series



Manual M/MK Series



NOVA class B

tuttnauer

Sterilization & Infection Control Products